



## **Test Update 683**

**Posted Date** 09/18/2019

**Effective Date** 09/18/2019

**Update Type** [New Tests](#)

**CPT Code** 82017

Order Code: ACARN

CPT Code: 82017

Fee Code: DA160

Effective October 1, 2019, MLabs will offer Acylcarnitines, Quantitative, Plasma (ACARN) to replace the Acylcarnitines, Quantitative, Plasma test sent to Mayo Medical Laboratories.

Specimen Collection: Collect approximately 3 mL of whole blood in a green top tube (sodium heparin). Centrifuge within 2 hours of collection, aliquot plasma and freeze (preferred) or store at refrigerated temperature (2-8°C) for up to 7 days. Grossly hemolytic specimens are unacceptable. Plasma specimens stored at room temperature greater than 2 hours are unacceptable.

Methodology: Liquid Chromatography-Tandem Mass Spectrometry (LC-MS/MS)

Set Up: Monday, Thursday

[View PDF](#)