



## [Test Update 874](#)

**Posted Date** 03/26/2024

**Effective Date** 03/22/2024

**Test Name** [Leptin](#)

**Update Type** [Test Methodology Changed](#)

**CPT Code** 83520

**EFFECTIVE DATE:** March 22, 2024

### **TEST METHODOLOGY UPDATE**

#### **Leptin**

Order Code: FLEP

Fee Code: AA082 (CPT 83520)

Reference Laboratory: Mayo FLEP (Esoterix Endocrinology)

Effective March 22, 2024, the test methodology for the Leptin assay referred to Esoterix Endocrinology has changed from Enzyme Immunoassay (EIA) to Enzyme-Linked Immunosorbent Assay (ELISA). This change has resulted in updates to specimen collection and handling and reference ranges as follows:

Specimen Collection: A fasting specimen is recommended (8-12 hour fast). Collect specimen in a red top or SST tube. Centrifuge and aliquot 1 mL of serum into a screw capped plastic vial within 45 minutes of collection. Freeze (preferred) or send at room temperature or refrigerated within 48 hours.

-

Reference Range: Adults (BMI=22): Male: 0.7 - 5.3 ng/mL, Female: 3.3 - 18.3 ng/mL. Please contact laboratory for other BMI reference ranges.

[View PDF](#)