

## **Test Update 924**

Posted Date 10/01/2025 Effective Date 10/16/2025 Test Name Beta 2 Transferrin

**Update Type** Specimen collection or handling and/or storage changed

**CPT Code** 86335

## **TEST CHANGE**

## **Beta 2 Transferrin**

Order Code: BETA2

Fee Code: AA727 (CPT 86335)

LOINC: 13876-8

Reference Lab: Mayo BETA2

Effective October 16, 2025, Mayo Clinic Laboratories will make Beta 2 Transferrin obsolete due to the availability of an automated methodology for Beta-Trace Protein. The recommended alternative test is Beta-Trace Protein, Body Fluid (Mayo test code: BTP)

<u>Collection Instructions</u>: Collect 0.5 mL of body fluid in a sterile container, syringe (with needle removed), test tube, or microtube. Plain cotton swab, pledget, or facial tissue is also acceptable. **Do not** collect specimen with a culture swab. **Do not** add any additional liquid other than source to the swab or gauze. **Do not** collect or send swab or gauze specimens in containers with additional liquids or additives

## Reference Range:

Beta-trace protein concentration < 5 mg/L is negative for cerebrospinal fluid.

Beta-trace protein concentration 5-7 mg/L is indeterminate for presence of cerebrospinal fluid.

Beta-trace protein concentration >7 mg/L is consistent with the presence of cerebrospinal fluid.

**View PDF**