



[Test Update 925](#)

Posted Date 10/08/2025

Effective Date 11/06/2025

Update Type [Test Discontinued](#)

CPT Code 80194

TEST OBSOLETE

Quinidine, Serum

Order Code: MQUIN

Fee Code: AA351 (CPT 80194)

Reference Lab: Mayo QUIN

Due to limited utilization, Mayo Clinic Laboratories will make Quinidine, Serum (test code QUIN) obsolete effective November 6, 2025. The recommended alternative will be forwarded to LabCorp (test: Quinidine, Serum or Plasma - 007831).

Collection Instructions: Collect a red-top or green-top (heparin) tube. **Do not use a gel-barrier tube.** Transfer separated serum or plasma into a plastic transport tube. Specimen is stable at room temperature, refrigerated, and frozen for 14 days.

Peak: quinidine sulfate: 1.5 hours after dose, quinidine gluconate: 4 hours after dose.

Trough: immediately prior to next dose.

After change in dose: one to two days.

Reference Range: Therapeutic: 2.0-5.0 µg/mL

[View PDF](#)