



Test Update 683

Posted Date 09/18/2019
Effective Date 09/18/2019
Update Type [New Tests](#)
CPT Code 82017

Order Code: ACARN
CPT Code: 82017
Fee Code: DA160

Effective October 1, 2019, MLabs will offer Acylcarnitines, Quantitative, Plasma (ACARN) to replace the Acylcarnitines, Quantitative, Plasma test sent to Mayo Medical Laboratories.

Specimen Collection: Collect approximately 3 mL of whole blood in a green top tube (sodium heparin). Centrifuge within 2 hours of collection, aliquot plasma and freeze (preferred) or store at refrigerated temperature (2-8°C) for up to 7 days. Grossly hemolytic specimens are unacceptable. Plasma specimens stored at room temperature greater than 2 hours are unacceptable.

Methodology: Liquid Chromatography-Tandem Mass Spectrometry (LC-MS/MS)

Set Up: Monday, Thursday

[View PDF](#)