



Test Update 724

Posted Date 07/22/2020

Effective Date 08/17/2020

Update Type [Specimen collection or handling and/or storage changed](#)

CPT Code 86940, 86941

Referral Laboratory: Mayo Medical Laboratories via ARUP

Effective August 17, 2020 Donath Landsteiner Test (MMLR) test requirements are changing per Mayo Medical Laboratories via ARUP.

Current Specimen Required

Draw blood in a plain red-top tube. Maintain specimen at 37 degrees C until serum is separated from cells. Spin down and send 2 mL of serum refrigerated in a plastic vial.

New Specimen Required

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 3 mL

Draw blood in a plain red-top tube. Maintain specimen at 37 degrees C until serum is separated from cells. Spin down and send **3 mL** of serum refrigerated in a plastic vial.

Current Requested Volume

2 mL

New Requested Volume

3 mL

[View PDF](#)