



## **Test Update 724**

**Posted Date** 07/22/2020

**Effective Date** 08/17/2020

**Update Type** [Specimen collection or handling and/or storage changed](#)

**CPT Code** 86940, 86941

**Referral Laboratory:** Mayo Medical Laboratories via ARUP

Effective August 17, 2020 Donath Landsteiner Test (MMLR) test requirements are changing per Mayo Medical Laboratories via ARUP.

### **Current Specimen Required**

Draw blood in a plain red-top tube. Maintain specimen at 37 degrees C until serum is separated from cells. Spin down and send 2 mL of serum refrigerated in a plastic vial.

### **New Specimen Required**

**Specimen Type: Serum**

**Container/Tube: Red**

**Specimen Volume: 3 mL**

Draw blood in a plain red-top tube. Maintain specimen at 37 degrees C until serum is separated from cells. Spin down and send **3 mL** of serum refrigerated in a plastic vial.

### **Current Requested Volume**

2 mL

### **New Requested Volume**

3 mL

[View PDF](#)